

INNOVATION FOR ALL



EXTRACELLULAR
MATRIX SUBSTITUTE

**Dermalpliq
Spray**


SKIN CARE FOR PETS



VET-EN-12/2017

- RGTA® exclusive regenerative technology
- Functional and economical
- Up to 6 months of use

Tissue Regeneration

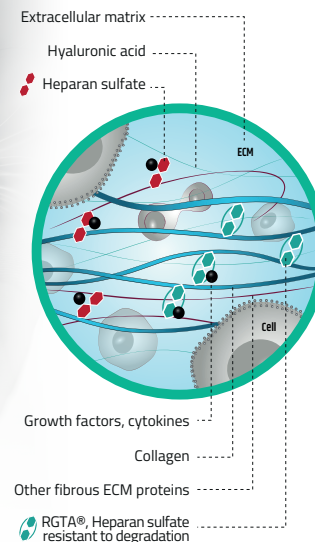

Sterile solution
for external use

Easy to use

Box of a 7.5 ml
spray bottle :
about 50 sprays

Without
preservative

3K® pump :
prevents the
entry of germs or
impurities into the
bottle

Manual

EXTERNAL USE

1. **Clean**
the wound.



2. **Spray
DERMAPLIQ**
onto the wound
from a distance
of 5 cm.



3. **Leave for
about 1 minute**
before applying
any other
product, or
applying a
dressing.



DERMAPLIQ SPRAY acts on living tissue. Apply
every 2 to 3 days depending on healing. It is
not recommended to use the product daily.
The duration of use can vary from one to
several weeks.



Dermalpliq Spray

IS SUITABLE FOR USE IN
CATS, DOGS, SPORT AND
RACE HORSES, AND EXOTIC
PETS

Learn more

- Barritault D, et al
"RGTA® or ReGeneraTing
Agents mimic heparan sulfate
in regenerative medicine: from
concept to curing patients."
Glycoconj J. 2017 Jun;34(3):
325-338
- "Utilisation des RGTA pour
la gestion des plaies – Cas
Cliniques"
Abstract Vet, supplément #22,
Sept 2015
- Zakine G, et al
"Matrix therapy with RGTA
OTR4120 improves healing
time and quality in hairless
rats with deep second-degree
burns."
Plast Reconstr Surg. 2011
Feb;127(2):541-50
- Groah, et al
"Regenerating matrix-based
therapy for chronic wound hea-
ling: a prospective within-sub-
ject pilot study."
Int Wound J. 2011 Feb; 8:85-95

www.otr3.com
contact@otr3.com